



LIAISON NES® FLU A/B, RSV & COVID-19 QUICK REFERENCE GUIDE - PREPARE FOR A PATIENT TEST AND RUNNING A PATIENT SAMPLE

CLIA Complexity: WAIVED*

*This statement only applies to customers within the United States:

For use with anterior nasal swab specimens.

A Certificate of Waiver is required to perform this test in a CLIA Waived setting. To obtain CLIA Waiver information and a Certificate of Waiver, please contact your state health department. Additional CLIA Waiver information is available at the Centers for Medicare and Medicaid website at www.cms.hhs.gov/CLIA.

Failure to follow the instructions or modifications to the test system instructions will result in the test no longer meeting the requirements for waived classification.

Materials Included in the Kit

Cartridge




Sample vial



Swab tube



Single test box

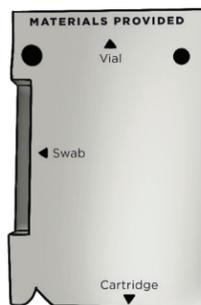

LIAISON NES® Instrument



A. Touch Screen **B.** Cartridge Bay **C.** Barcode Scanner Window

Before performing this test, refer to the LIAISON NES® FLU A/B, RSV & COVID-19 Instructions for Use for the complete Test Procedure and the LIAISON NES® User Manual for additional information.

NOTE: On the first use of this device, Quality Controls (positive and negative controls) must be run as a first step; refer to the LIAISON NES® FLU A/B, RSV & COVID-19 QUICK REFERENCE GUIDE - PREPARING FOR A QUALITY CONTROL TEST.


All LIAISON NES® Kit disposable components are designed for single use only and must not be reused.

Do Not remove or tamper with the foil on the underside of the cartridge.


Do Not use any components after the printed expiration date.

To perform a patient test, follow the directions below.

1 PREPARE TO PERFORM THE TEST

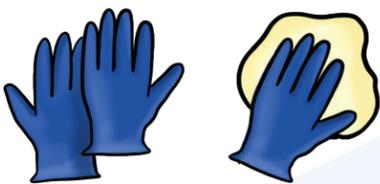


Wear new nitrile or latex gloves and appropriate personal protective equipment (PPE). Sanitize the work surface.

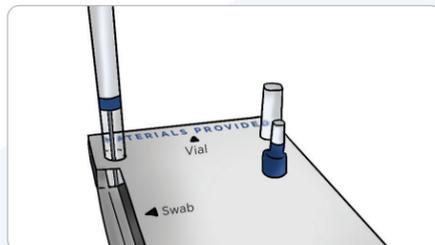


Materials set-up in single test box.

2 SAMPLE COLLECTION

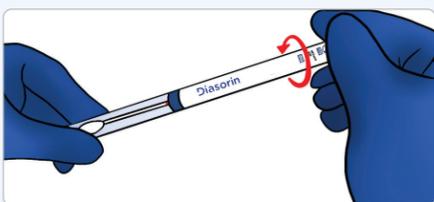

Twist open the swab tube.

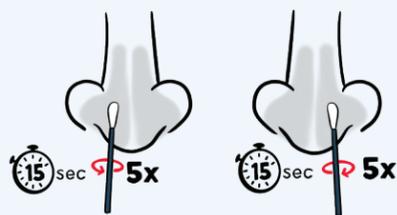

Carefully insert the nasal swab approximately 1 inch into one nostril. Gently rotate the swab **5 times** against the nasal wall for approximately **15 seconds**. Repeat in other nostril.

WARNING!
Inaccurate test results may occur if the nasal swab specimen is not properly collected.




After collecting the sample, return the swab back to the tube.


3 SAMPLE PREPARATION



Remove the foil seal from the sample vial.

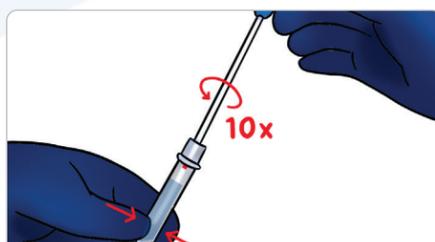

Place the swab in the sample vial and rotate the swab **10 times while squeezing** the swab head.

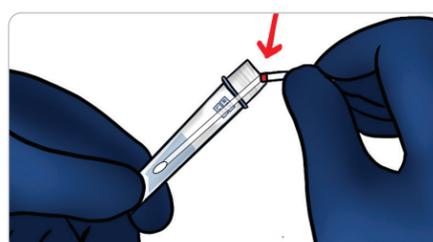

Break the swab shaft at the red mark by snapping along the rim of the sample vial.




Twist blue cap onto the sample vial with the swab head inside.

Dispose of swab handle into the biohazardous waste.

WARNING!
If any liquid spills or if a vial is leaking or empty - DO NOT USE. Use a new sample vial.

Diasorin

4 CARTRIDGE PREPARATION

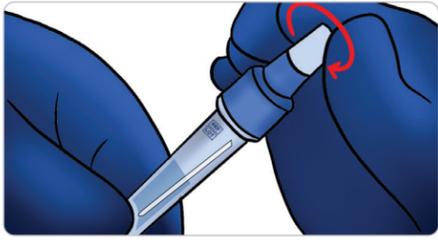

Unscrew the clear nozzle cap.



WARNING!
Place cartridge on hard flat surface and keep level.



Insert the tip of the sample vial into the cartridge sample port. Firmly squeeze the sample vial to dispense **ALL** the liquid into the cartridge.



Carefully remove the sample vial and close the cartridge lid securely.
Dispose of the sample vial and clear nozzle cap into the biohazardous waste.


Failure to transfer ALL the liquid into the cartridge can cause invalid results.




WARNING!

Keep cartridge upright - do not tilt or shake. Perform the test immediately.


5 RUNNING THE TEST

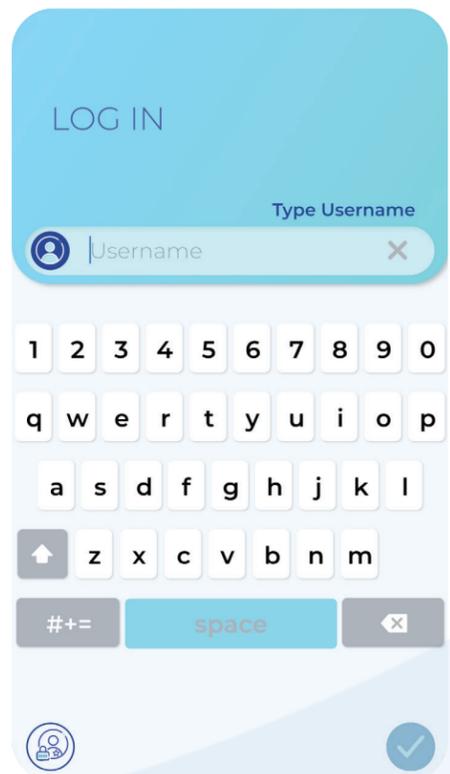

Login to the LIAISON NES® instrument.

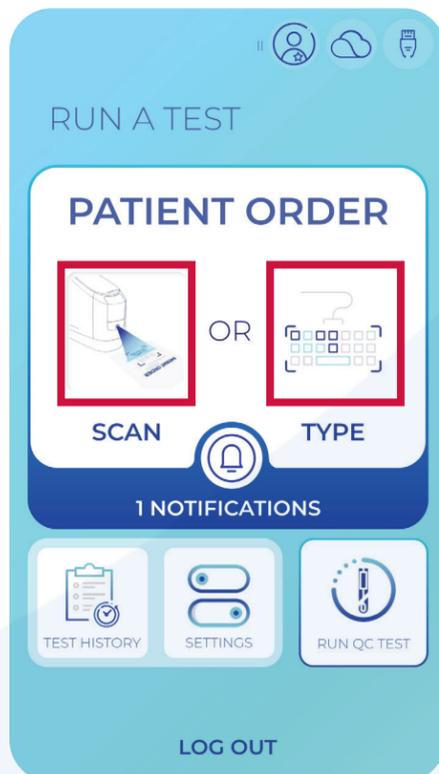


Scan or type in Patient ID. The instrument door will automatically open.

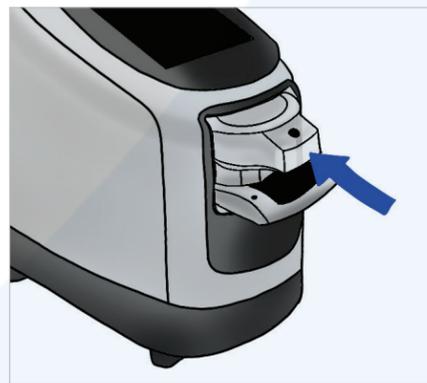

Ensure the cartridge is labeled with the sample ID and loaded. Fully close the door. Run the test following on-screen instructions.

6 COMPLETING THE TEST

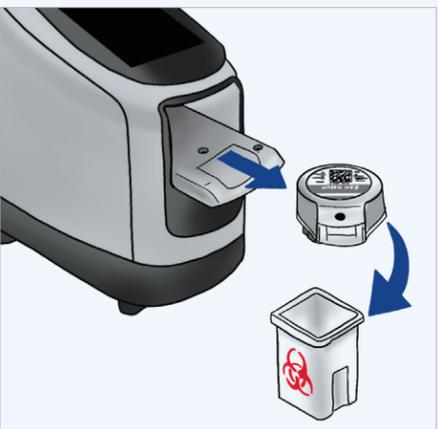

Door automatically opens, remove and dispose of the cartridge.

+	COVID-19	Positive
-	COVID-19	Negative
⊘	COVID-19	Invalid

CLOSE the door to view the test results.
In case of an invalid result, retest.

Diasorin

Cypress, CA
p: 800 838 4548
w: diasorin.com

diasorin.com/liaison-nes

CLIA COMPLEXITY: WAIVED

Refer to the LIAISON NES® FLU A/B, RSV & COVID-19 Assay Kit Instructions for Use and LIAISON NES® User Manual for complete instructions.

Rx Only. For In Vitro Diagnostic Use. Products are region specific and may not be approved in some countries/regions.

©2025 DiaSorin Molecular LLC. All rights reserved. LIAISON NES is a trademark of DiaSorin Molecular LLC and/or its affiliates, registered in the US and other countries.