




LIAISON NES® GROUP A STREP QUICK REFERENCE GUIDE -
PREPARING FOR A QUALITY CONTROL TEST

CLIA Complexity: **WAIVED***

*This statement only applies to customers within the United States:
For use with throat swab specimens.
A Certificate of Waiver is required to perform this test in a CLIA Waived setting. To obtain CLIA Waiver information and a Certificate of Waiver, please contact your state health department. Additional CLIA Waiver information is available at the Centers for Medicare and Medicaid website at www.cms.hhs.gov/CLIA.
Failure to follow the instructions or modifications to the test system instructions will result in the test no longer meeting the requirements for waived classification.

Materials Needed from Control Swab Kit
and Single Test Box

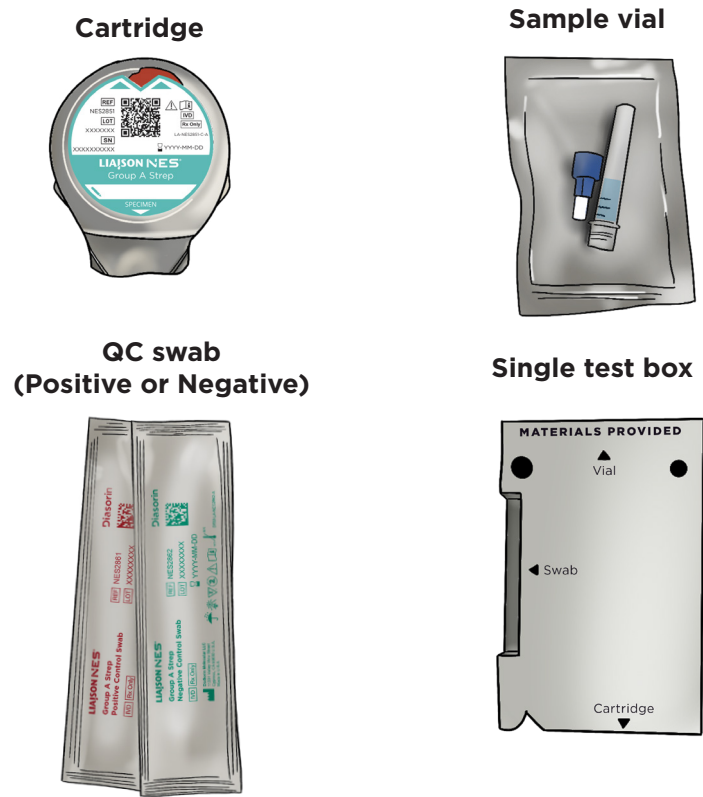

LIAISON NES® Instrument



A. Touch Screen **B.** Cartridge Bay **C.** Barcode Scanner Window

Before performing this test, refer to the LIAISON NES® Group A Strep Instructions for Use for the complete Test Procedure and the LIAISON NES® User Manual for additional information.

NOTE: Run **BOTH** a positive and negative QC.


All LIAISON NES® Kit disposable components are designed for single use only and must not be reused.

Do Not remove or tamper with the foil on the underside of the cartridge.


Do Not use any components after the printed expiration date.

To perform a QC test, follow the directions below.

1 PREPARE TO PERFORM THE QC TEST

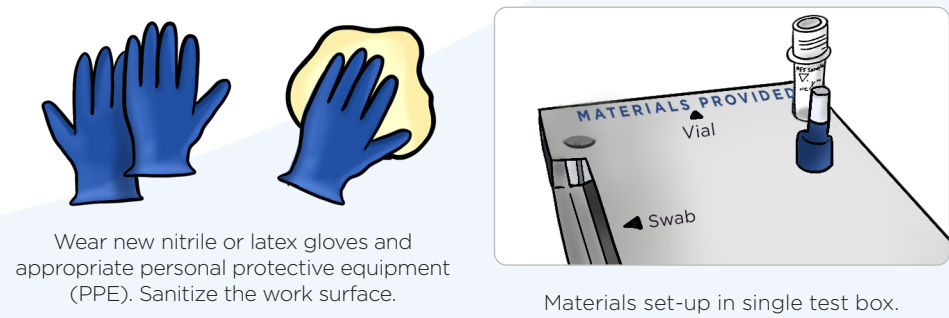


2 PREPARATION OF QC SWAB

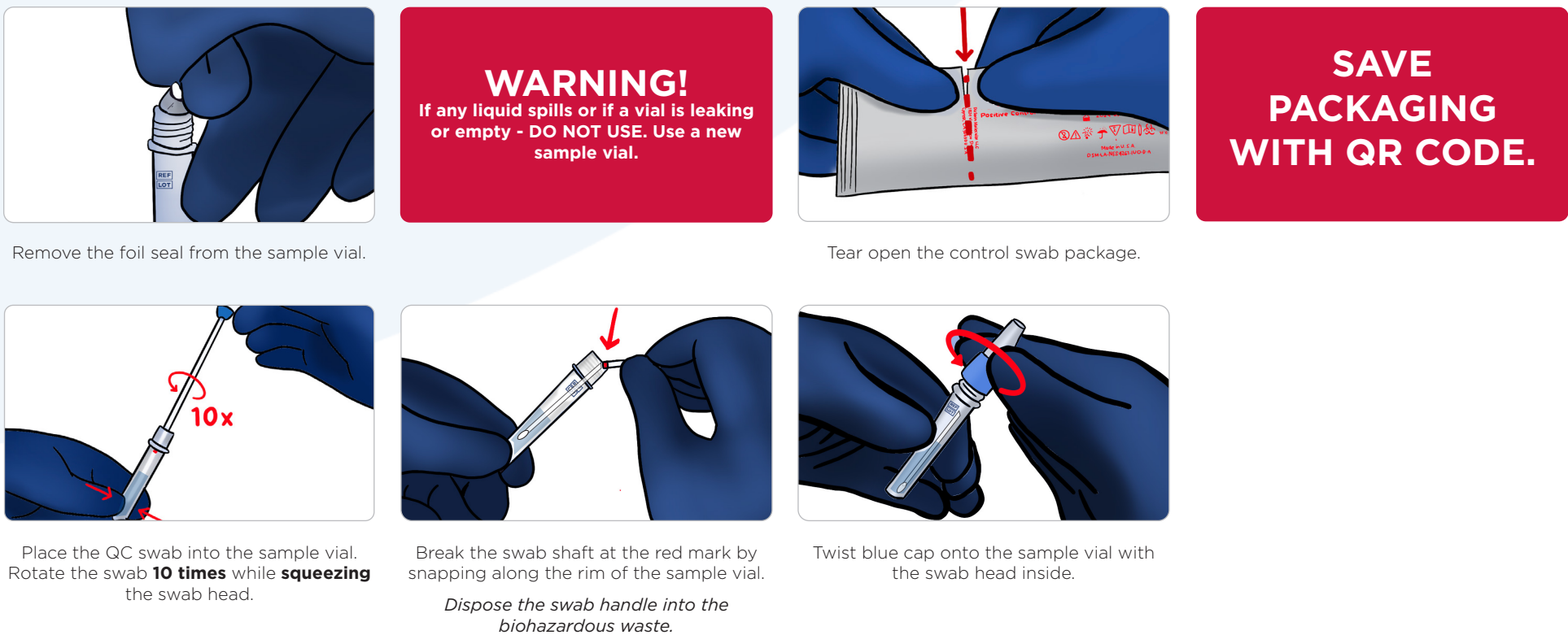

Diasorin

3 CARTRIDGE PREPARATION

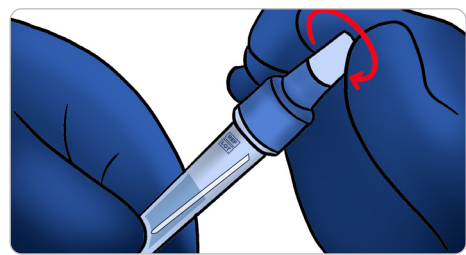

Unscrew the clear nozzle cap.



WARNING!

Place cartridge on hard flat surface and keep level.



Insert the tip of the sample vial into the cartridge sample port. Firmly squeeze the sample vial to dispense **ALL** the liquid into the cartridge.

Failure to transfer ALL the liquid into the cartridge can cause invalid results.



Carefully remove the sample vial and close the cartridge lid securely.


Dispose the sample vial and clear nozzle cap into the biohazardous waste.





WARNING!

Keep cartridge upright - do not tilt or shake. Perform the test immediately.

4 START QC TEST


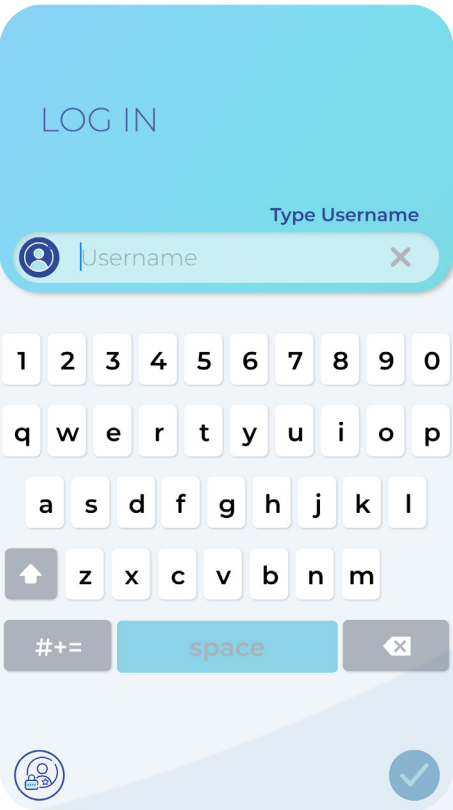


Login to the LIAISON NES® instrument.

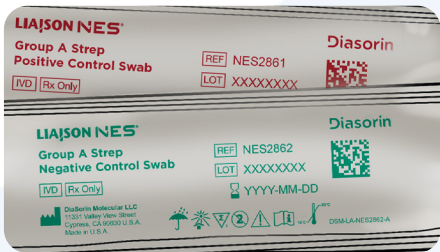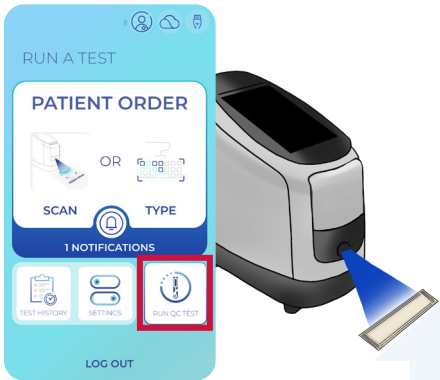


Press "Run QC" Test from the home screen.

Scan the QR Code from the emptied control packaging. The instrument door will automatically open.

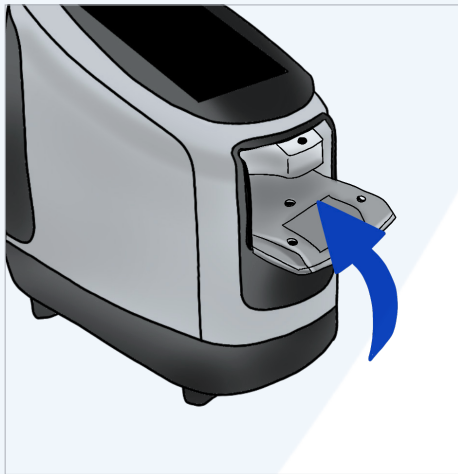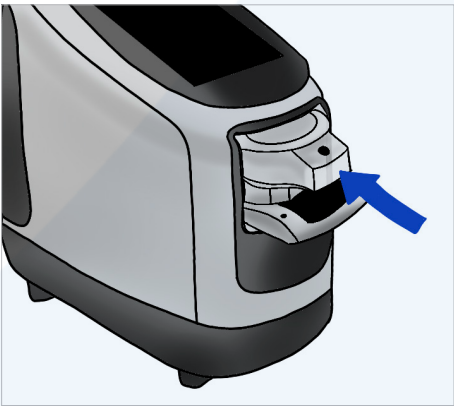

Ensure cartridge is labeled with control type and loaded. Run the test following on-screen instructions.

5 COMPLETING THE TEST

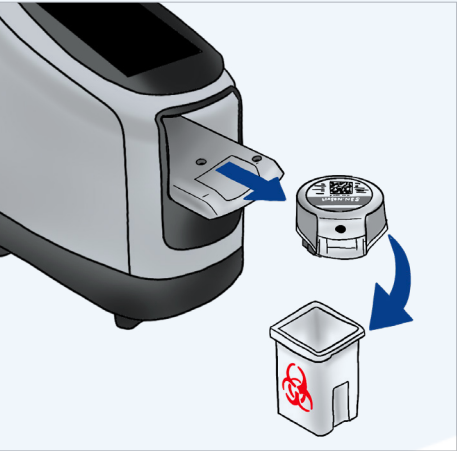

Door automatically opens, remove and dispose the cartridge.



Repeat QC test for the other control swab.

CLOSE the door to view results. In case of a **FAIL**, retest.

Diasorin

Cypress, CA
p: 800 838 4548
w: diasorin.com

diasorin.com/liaison-nes
CLIA COMPLEXITY: WAIVED

Refer to the LIAISON NES® Group A Strep Control Swab Kit Instructions for Use and LIAISON NES® User Manual for complete instructions.

Rx Only. For In Vitro Diagnostic Use. Products are region specific and may not be approved in some countries/regions.

©2026 DiaSorin Molecular LLC. All rights reserved. LIAISON NES is a trademark of DiaSorin Molecular LLC and/or its affiliates, registered in the US and other countries.