




LIAISON NES® GROUP A STREP QUICK REFERENCE GUIDE -
PREPARING FOR A PATIENT TEST AND RUNNING A PATIENT SAMPLE

CLIA Complexity: **WAIVED***

*This statement only applies to customers within the United States:
For use with throat swab specimens.
A Certificate of Waiver is required to perform this test in a CLIA Waived setting. To obtain CLIA Waiver information and a Certificate of Waiver, please contact your state health department. Additional CLIA Waiver information is available at the Centers for Medicare and Medicaid website at www.cms.hhs.gov/CLIA.
Failure to follow the instructions or modifications to the test system instructions will result in the test no longer meeting the requirements for waived classification.

Materials Included in the Kit

LIAISON NES® Instrument

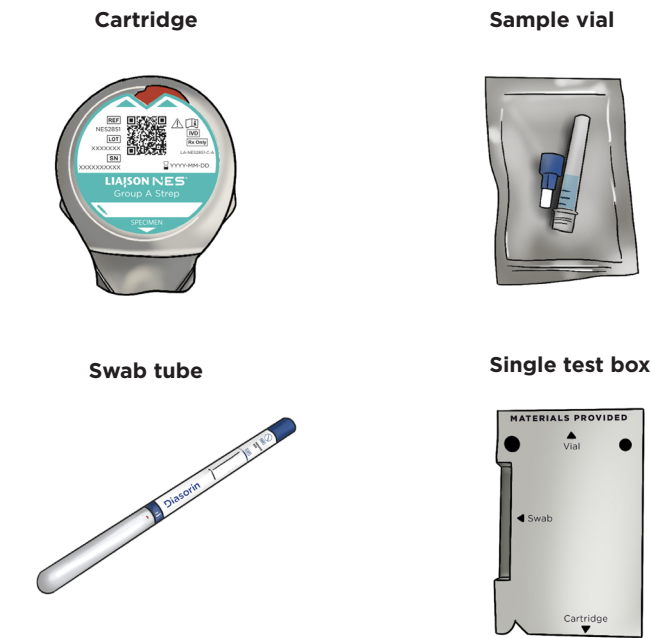


All LIAISON NES® Kit disposable components are designed for single use only and must not be reused.
Do Not remove or tamper with the foil on the underside of the cartridge.
Do Not use any components after the printed expiration date.


A. Touch Screen **B.** Cartridge Bay **C.** Barcode Scanner Window
Before performing this test, refer to the LIAISON NES® Group A Strep Instructions for Use and LIAISON NES® User Manual for complete Test Procedure and additional information.
NOTE: On the first use of this device, Quality Controls (positive and negative controls) must be run as a first step; refer to the LIAISON NES® GROUP A STREP QUICK REFERENCE GUIDE - PREPARING FOR A QUALITY CONTROL TEST.

To perform a patient test, follow the directions below.


1 PREPARE TO PERFORM THE TEST

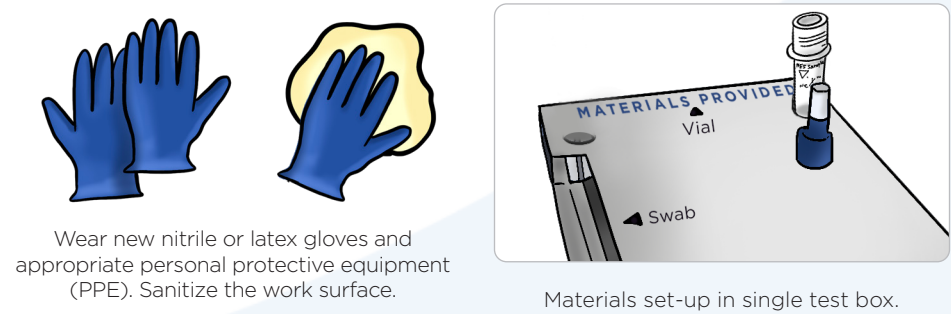

2 SAMPLE COLLECTION

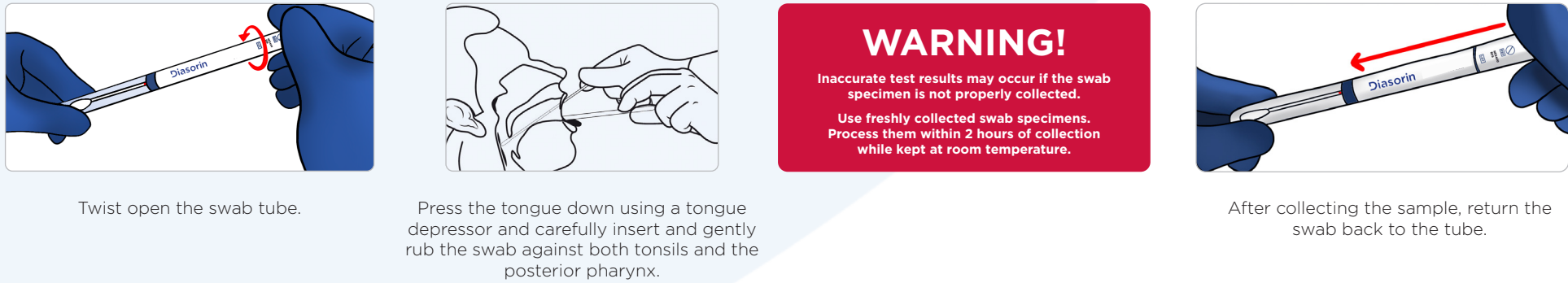

3 SAMPLE PREPARATION

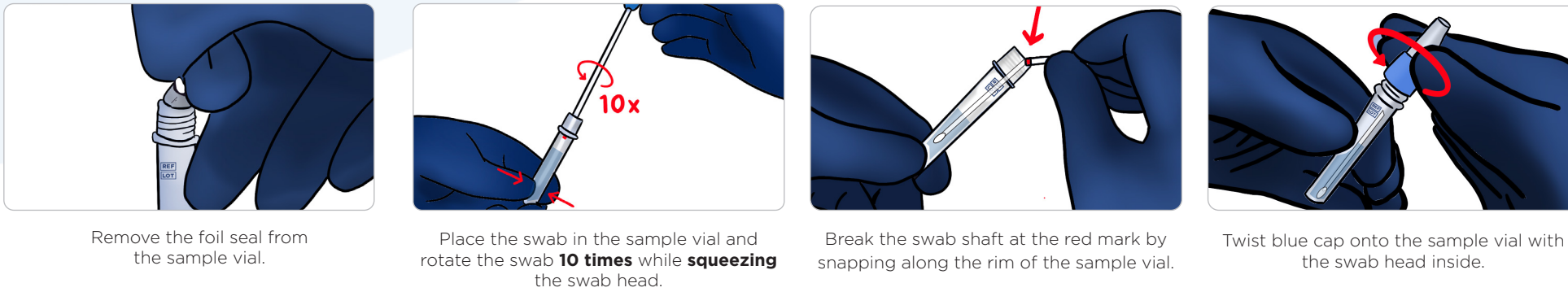

WARNING!
If any liquid spills or if a vial is leaking or empty - DO NOT USE.
Use a new sample vial.

Dispose swab handle into the biohazardous waste.

Diasorin

4 CARTRIDGE PREPARATION



Unscrew the clear nozzle cap.



WARNING!
Place cartridge on hard flat surface and keep level.



Insert the tip of the sample vial into the cartridge sample port. Firmly squeeze the sample vial to dispense **ALL** the liquid into the cartridge.




Carefully remove the sample vial and close the cartridge lid securely.
Dispose the sample vial and clear nozzle cap into the biohazardous waste.




WARNING!
Keep cartridge upright - do not tilt or shake. Perform the test immediately.


5 RUNNING THE TEST

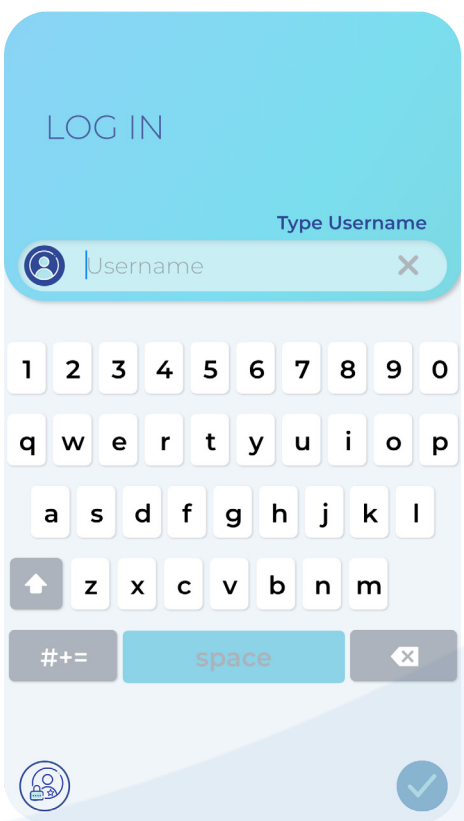

Login to the LIAISON NES® instrument.

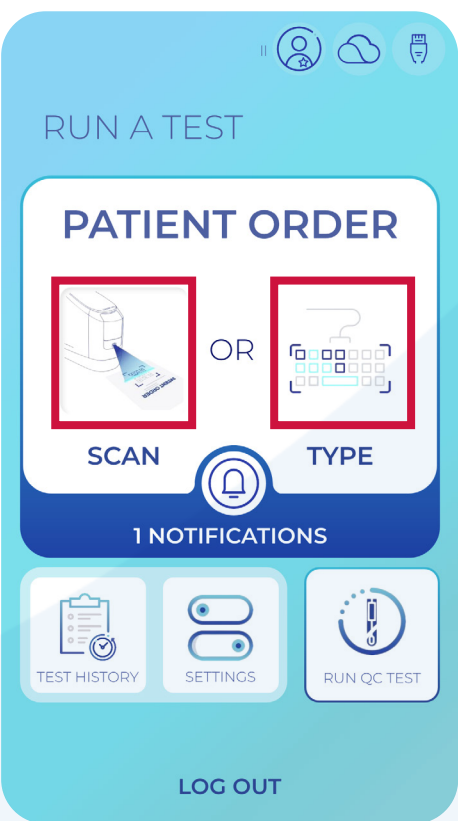


Scan or type in Patient ID. The instrument door will automatically open.

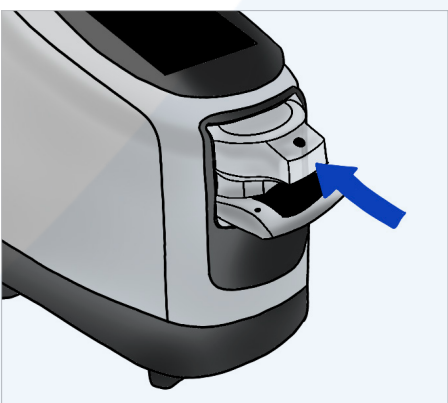

Ensure the cartridge is labeled with the sample ID and loaded. Fully close the door.
Run the test following on-screen instructions.

6 COMPLETING THE TEST

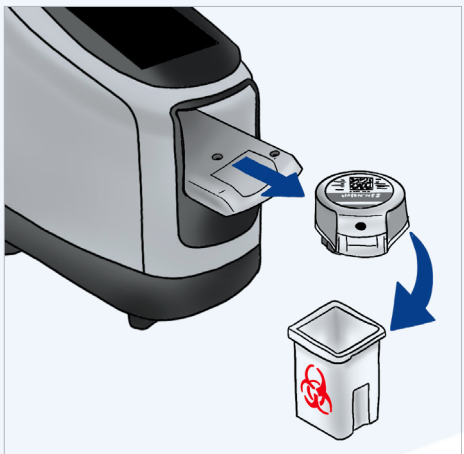

Door automatically opens, remove and dispose the cartridge.

+	Strep A	Positive
—	Strep A	Negative
⊘	Strep A	Invalid

CLOSE the door to view the test results.
In case of an invalid result, retest.

Diasorin
Cypress, CA
p: 800 838 4548
w: diasorin.com

diasorin.com/liaison-nes
CLIA COMPLEXITY: WAIVED

Refer to the LIAISON NES® Group A Strep Instructions for Use and LIAISON NES® User Manual for complete instructions.
Rx Only. For In Vitro Diagnostic Use. Products are region specific and may not be approved in some countries/regions.

©2026 DiaSorin Molecular LLC. All rights reserved. LIAISON NES is a trademark of DiaSorin Molecular LLC and/or its affiliates, registered in the US and other countries.