



Scan the QR code for animated instructions and customer support

## LIAISON NES® QUICK START GUIDE

### Unpack the LIAISON NES®

1. Unpack the LIAISON NES® instrument and check the instrument for any visible damage. Report any damage to Diasorin immediately. (See **Figures 2 & 3**)
2. Place the instrument on a clean, flat, stable surface near an electrical outlet.
3. Choose a location where the air vents on the back of the instrument (**Figure 3**) will remain unobstructed and the instrument is out of direct sunlight.

### Power on the LIAISON NES®

4. Attach the power cable by aligning the connector to the Power Connection Port on the back of the instrument (**Figure 1**), with the flat side facing inward.
5. Insert the power cable plug into an electrical outlet.
6. Turn on the instrument using the Power Switch on the back of the instrument (**Figure 3**).



Figure 1: Power Connection Port

### Setup the LIAISON NES®

7. Follow the prompts on the Touch Screen (**Figure 2**) to complete the guided setup.
8. Enter the date and time when prompted.
9. Create an “Administrator” account by entering the required details.

**Note:** Administrator users have access to the Settings menu. Operator users do not.

10. Connect the instrument to your network by plugging an Ethernet cable into the Ethernet Port (**Figure 3**). If Ethernet is not available, follow the on-screen instructions in the connectivity menu to connect to Wi-Fi.



Figure 2: Front of the instrument



Figure 3: Back of the instrument